




| Task | Resource | How Measured? | Achieved? | | N/A | Date | Trainee Initial | Trainer Initial |
|---|---|--|--------------------------|--------------------------|--------------------------|------|-----------------|-----------------|
| | | | Y | N | | | | |
| IDENTIFY POTENTIAL HAZARDS AND SAFETY FEATURES | User's Manual | | | | | | | |
| Review MSDS | CellaVision Immersion Oil MSDS BCI Website | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Recognize Hazard Symbols and wear proper PPE | CellaVision User's Manual & Lab SOP | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| IDENTIFY AND BRIEFLY EXPLAIN FUNCTION OF SYSTEM COMPONENTS | CellaVisions User's Manual | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| OPERATION PROCEDURES | CellaVision User's Manual | | | | | | | |
| Demonstrate how to log into software and correct database | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Start up the SSU and DM System Computer | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Properly load barcoded slides into magazine | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Load and process magazine on SSU | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| | | | | | | | | |

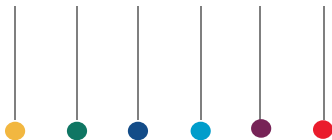


| Task | Resource | How Measured? | Achieved? | | N/A | Date | Trainee Initial | Trainer Initial |
|---|---------------------------|--|--------------------------|--------------------------|--------------------------|------|-----------------|-----------------|
| | | | Y | N | | | | |
| Explain buttons and indicators of System Control View | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Eject magazine | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate processing of STAT sample | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Shutdown the SSU and DM System Computer | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| QUALITY CONTROL-CELL LOCATION | CellaVision User's Manual | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Explain importance of Cell Location testing | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Prepare a Cell Location peripheral blood slide | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Process a Cell Location slide and verify slide is within performance specifications | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |

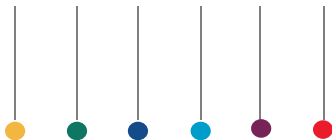


| | | | | | | | | |
|--|---------------------------|--|--------------------------|--------------------------|--------------------------|--|--|--|
| Prepare a Cell Location body fluid slide (if applicable) | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Process a body fluid Cell Location slide and verify slide is within performance specifications (if applicable) | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Explain how often Cell Location testing needs to be completed on peripheral bloods and body fluids | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| VERIFY PROCESSED SLIDES | CellaVision User's Manual | | | | | | | |
| Explain buttons and indicators of the Database View screen | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Locate and open a processed slide and an unsigned slide | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| View WBC classifications in one, two, three cell galleries and full screen view | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Complete reclassifications of WBCs | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| | | | | | | | | |

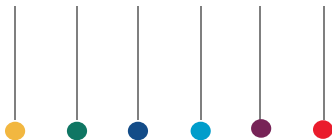

| | | | | | | | | |
|---|---------------------------|--|--------------------------|--------------------------|--------------------------|--|--|--|
| Complete morphology classifications of RBC precharacterizations | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Complete a PLT estimate | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Explain different types of comments and add comments to WBC, RBC, and PLT results | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate how to mark a slide for Path review (if applicable) | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate how to edit patient order data | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Sign off a processed slide | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| REPORTING RESULTS | CellaVision User's Manual | | | | | | | |
| Demonstrate how to merge multiple slide orders | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Explain the components of a Patient Report | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate how to sign orders | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| | | | | | | | | |

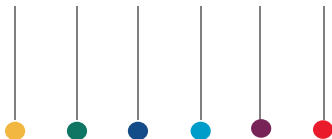

| | | | | | | | | |
|--|---------------------------|--|--------------------------|--------------------------|--------------------------|--|--|--|
| Demonstrate how to send an order to the LIS and how to cancel an Order | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| MAINTENANCE | CellaVision User's Manual | | | | | | | |
| Demonstrate how to clean the XY Stage | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate how to clean the objectives and explain the importance of keeping the 10x objective oil free | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate how to clean and oil the Slide Guides (SN 31001-31215) | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Restart the DM System Computer and SSU | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Delete unsigned/failed orders | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate how to exchange the bulb | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Explain database maintenance and how to archive/autodelete | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate how to compress a database (if applicable) | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |

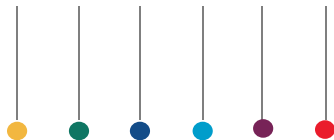

| | | | | | | | | |
|--|---------------------------|--|--------------------------|--------------------------|--------------------------|--|--|--|
| Demonstrate how to refill the immersion oil | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Explain and demonstrate how to remove a stuck slide | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| MISCELLANEOUS | CellaVision User's Manual | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Prepare a slide based on slide and stain guidelines | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate how to use Remote Review software (if applicable) | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Create a pending order for a peripheral blood slide and body fluid slide (if applicable) | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |
| Demonstrate how to export an order (if applicable) | | <input type="checkbox"/> Direct Observation <input type="checkbox"/> Test <input type="checkbox"/> Other _____ | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | | | |