

iRICELL Maintenance

<u>Daily</u>
Aliquot Velocity QC (CA,CB,CC)
Check urine strip supply
Check volume in Wash Container
Check volume in Lamina Container
Clean exterior of system
Clean the sample transport module
Clean optional load/unload trays
Clean strip waste container
Clean liquid waste bottle, if applicable
Run iQ cleaning and QC rack
Run Velocity QC rack

<u>Quarterly</u>
Velocity Reflectance Check
Velocity CGM Check
Wash Solution, if needed

<u>As Needed</u>
Clean sample tube detectors (both)
Clean bar code readers (both)
Clean sample transport sensors (both)
Clean iQ Rinse Wash Bath
Replace Lamina
Flush iQ Sample Filter

<u>Weekly</u>
Clean Strip Provider Module (SPM)
Clean Spec. Gravity/Color/Clarity Module (CGM)

<u>Monthly</u>
Perform iQ Calibration
Clean Velocity Wash Station
Clean the Strip Conveyor System (SCS)
Backup the system

Frequency	iChem Velocity	iQ 200
Daily	√	√
Weekly	√	
Monthly	√	√
Quarterly	√	
As-Needed	√	√