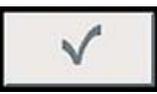


## Power Express Module Control Panel Buttons

BUTTON	DESCRIPTION
	<b>Auto:</b> Select to place the module in Auto (RUN) mode. In Auto mode, the module processes sample tubes without user intervention
	<b>Manual:</b> Select to place the module in Manual (STOP) mode. You must place the module in Manual mode before you perform any maintenance functions
	<b>Maintenance:</b> Access the Maintenance functions for the module
	<b>Pause:</b> Select to temporarily PAUSE the module
	<b>Restart:</b> Select to start the module after a PAUSE, error release, or rack exchange.
	<b>Rack Exchange:</b> Select to perform a rack exchange at the Inlet, Outlet, or Storage module
	<b>Left Drawer:</b> Select to unlock the left Inlet or Outlet drawer to perform a rack exchange or to perform a rack exchange at the Storage module
	<b>Right Drawer:</b> Select to unlock the right Inlet or Outlet drawer to perform a rack exchange
<b>The following icons only display during a Rack Exchange at the Outlet and Storage module control panels:</b>	
	<b>Rack Exchange Not Required:</b> Indicates the rack is not full; no rack exchange needed for this position
	<b>Rack Exchange Required:</b> Indicates the full rack position requiring an exchange
	<b>Rack Exchange In Progress:</b> Indicates the rack has been removed, but the empty rack has not been loaded
	<b>Rack Exchange Completed:</b> Indicates the rack has been removed and replaced with an empty rack. The system will clear all positions with this icon when <b>Restart</b> is selected <b>NOTE:</b> To cancel the clearing of a rack, select the position so that it changes back to the <b>Rack Exchange Not Required</b> icon (white box).

## Power Express Module Control Panel Buttons

BUTTON	DESCRIPTION
	<b>Mute:</b> Select to silence the alarm when an error occurs or during a rack exchange
	<b>Help:</b> Select to display a brief description of the error
	<b>Unlock:</b> Select to PAUSE the module and unlock the front door or cover. For the Centrifuge module, press and hold this button to unlock the centrifuge lid
	<b>Top Unlock:</b> For the Centrifuge module, select to PAUSE the module and unlock the centrifuge cover. You must then select the <b>Unlock</b> button to unlock the centrifuge lid
	<b>Enter:</b> Select to enter your information into the system
	<b>Exit:</b> Select to return to the previous screen
	<b>Priority:</b> Select to override Load Timeout and start operation immediately. This button applies to the Centrifuge and Rack Builder modules, and to the Level Detector component of the Aliquot module
	<b>Clear (error release):</b> If you need to remove a sample tube that has caused an error, select to bypass the sample tube, clear the error, and restart the module
	<b>Bypass:</b> Select to bypass the module
	<b>Repeat:</b> Select to repeat the action of the function code you entered
	<b>Yes/OK:</b> Select to confirm your selection or changes
	<b>No/Cancel:</b> Select to cancel your selection or changes